		1	
Yester Yester Big Note Big Service Bi	PICO° dressings are designed for use with the PICO single use negative pressure wound therapy system. For any information on the PICO systems refer to the respective PICO system user manual. PICO is intended for use by or on the direction of a trained and licensed physician in accordance with the PICO system user manual.	 1. Description PICO dressings are packaged sterile and are for single-use only. The PICO dressing manages exudate through a combination of absorption and evaporation of moisture through the outer film. The PICO dressing is intended to be used for up to 7 days on low exuding wounds. For the treatment of moderate exuding wounds the dressing is intended to be used for up to 4 days. Low exuding wounds are considered to be up to 0.6g of liquid exudate/cm² of wound area/24 hours. Moderate exuding wounds are considered to be up to 1.1g of liquid exudate/cm² of wound area/24 hours. Ig of exudate is approximately equal to 1ml of exudate. The frequency of dressing changes can be affected by multiple factors such as wound type, wound size, rate or volume of exudate, orientation or environmental conditions. 	 2. Indications for use PICO single use negative pressure sy indicated for patients who would bene suction device (negative pressure would they may promote wound healing via moderate levels of exudate and infect Appropriate wound types include: Chronic Acute Traumatic Subacute and dehisced wounds Partial-thickness burns Ulcers (such as diabetic or pressure sy for use both in a hospital and homecand second secon
<section-header><section-header><image/><section-header><text><text><text><text><text></text></text></text></text></text></section-header></section-header></section-header>	<section-header> 5. Warnings Magnet Warning The PICO 7 pump contains a MAGNET that an cause other medical devices in close proximity to fail, leading to serious harm including death. The PICO 7 pump must be positioned at least 4 inches (10cm) away from other medical devices that could be affected by magnetic interference. These include but are not limited to: Magnet Warning Insulin Pumps Shunt Valves Neurostimulators Cochlear implants THS WARNING APPLES AT ALL TIMES TO ALL USERS. This applies to both Patients and Caregivers. Unust keep the PICO 7 pump at least 4 inches (10cm) away from other devices. Insulin Pumps Shunt Valves Magnet Warning THS WARNING APPLES AT ALL TIMES TO ALL USERS. This applies to both Patients and Caregivers. You must keep the PICO 7 pump at least 4 inches (10cm) away from other devices. If you have an electronic medical device and are helping take care of somebody else using the PICO 7 system. If the patient is wearing the PICO 7 pump in a public are avhere they may come in close contact with someone else who has an electronic medical device. </section-header>	 Certain patients are at high risk of bleeding complications which, if uncontrolled, could potentially be fatal. Patients must be closely monitored for bleeding. If sudden or increased bleeding is observed, immediately disconnect pump, leave dressing in place, take appropriate measures to stop bleeding and seek immediate medical assistance. Hemostasis must be achieved before applying the dressing, although the use of anticoagulants does not deem a patient inappropriate for treatment with PICO. Patients suffering from difficult hemostasis or who are receiving anticoagulant therapy have an increased risk of bleeding. During therapy, avoid using hemostatic products that may increase the risk of bleeding, if disrupted. Frequent assessment must be maintained throughout the therapy. At all times care should be taken to ensure that the pump, tubing and connectors do not: Lie in a position where it could cause 	 pressure damage to the patie Trail across the floor where it of trip hazard or become contain Present a risk of strangulation to patients. Rest on or pass over a source Become twisted or trapped ur bandages so that therapy is b Sharp edges or bone fragments must be covered or removed price PICO due to risk of puncturing or blood vessels while under negat wound therapy. In the event that defibrillation is r the dressing if it is positioned in will interfere with defibrillation. PICO has not been studied on per Patient size and weight should b when prescribing this therapy. The system contains small parts represent a choking hazard for years.
<section-header><section-header><list-item><list-item><list-item><list-item><list-item></list-item></list-item></list-item></list-item></list-item></section-header></section-header>	 is important to visually inspect the system regularly, especially in the first week of treatment to ensure that negative pressure wound therapy is continually applied and a seal is maintained. 4. Where PICO dressings are used on infected wounds, more frequent dressing changes may be required. Regular monitoring of the wound should be maintained to check for signs of infection. 5. If deemed clinically appropriate, care should be taken that the application of a circumferential dressing or the use of negative pressure wound therapy on ischemic limbs does not compromise circulation. 6. Although PICO dressings can be used under clothing/bedding, it is important that occlusive materials e.g. film dressings, are not applied over the pad area of the dressing as this will impair the intended evaporation of moisture through its outer layer. 7. The PICO dressing should not be covered by rigid immobilization devices or casts which might apply 	 excessive pressure and cause tissue injury at the wound site, especially where the tubing enters the dressing. 8. Prolonged placement of rigid or opaque materials over the PICO dressing may prevent the regular inspection and assessment of the wound, and disrupt scheduled or required dressing changes. 9. Where PICO dressings are used on patients with fragile skin, a skin protectant such as NO-STING SKIN-PREP° should be used on areas of skin where fixation strips are to be applied. Inappropriate use or repeated application of fixation strips may otherwise result in skin stripping. 10. Do not use PICO dressings with oil-based products such as petrolatum as it may compromise establishing an effective seal. 11. The use of negative pressure wound therapy presents a risk of tissue ingrowth into foam when this is used as a wound filler. When using foam filler with PICO, tissue ingrowth may be reduced by using a non-adherent wound contact layer or by 	 increasing the frequency of dressin 12. PICO may be used in conjunction drains provided the dressing is not tubing where it exits the skin. Any should be routed under the skin a edge of the dressing and function of the PICO System. 13. When showering the PICO pump be disconnected from the dressin disconnected, ensure the end of attached to the dressing is facing water does not enter the tube. 14. The PICO dressing should only be PICO pumps. 15. Do not alter or cut tubing configurate tubing or soft port. 16. Do not cut the PICO dressing parlead to loss of negative pressure application. 17. Always ensure that the PICO dressing of the PICO dressing of the PICO dressing of the PICO dressing of the positioned centrally over the work.

	3. Contraindications	
systems are enefit from a vound therapy) as ia removal of low to ectious materials. sure) ystems are suitable care setting.	 PICO is contraindicated for: Patients with malignancy in the wound bed or margins of the wound (except in palliative care to enhance quality of life). Previously confirmed and untreated osteomyelitis. Non-enteric and unexplored fistulas. Necrotic tissue with eschar present. Exposed arteries, veins, nerves or organs. Exposed anastomotic sites. PICO should not be used for the purpose of: Emergency airway aspiration. Pleural, mediastinal or chest tube drainage. Surgical suction. 	
Page 3 of 29	Page 4 of 29	
ient. it could present a aminated. on or a tourniquet ce of heat. under clothing or blocked. s in a wound rior to using organs or ative pressure s required, remove n a location that pediatric patients. be considered ts which could young children.	<list-item><list-item><list-item><list-item><list-item></list-item></list-item></list-item></list-item></list-item>	
sing changes. on with surgical not placed over ny surgical drain n away from the on independently np should using. Whilst of the tubing ng down so that be used with guration or pull on	 port should be positioned uppermost on intact skin and not extend over the wound so that the risk of fluid collecting around the soft port and potentially blocking the therapy is minimised. 18. The PICO system is single use only. Use of any part of this system on more than one patient may result in cross contamination that may lead to infection. 19. High temperatures and humidity may reduce wear times of PICO dressings. 20. When applying dressings next to one another, ensure the dressing borders do not overlap. 	
bad as this may re wound therapy ressing is ound. The soft		

7. Adverse Reactions

Excessive bleeding is a serious risk associated with the application of suction to wounds which may result in death or serious injury. Careful patient selection, in view of the above stated contraindications, warnings and precautions is essential. Carefully monitor the wound and dressing for any evidence of a change in the blood loss status of the patient. Notify the healthcare professional of any sudden or abrupt changes in the volume or the color of exudate.

8. Instructions for use

8.1. Guidance on wound suitability

PICO dressings should be used on wounds which fit comfortably within the area of the pad, observing precautions on soft port positioning (on intact skin and not extending over the wound). PICO Multisite dressings are designed to enhance conformability when dressing awkward anatomical areas. Each PICO dressing (including Multisite) must be used to dress one wound only. As a guide:

Depth – Wounds greater than 0.5cm (1/4 in.) in depth are likely to require a foam or gauze negative pressure wound therapy filler to ensure adequate treatment of all the wound surfaces. Wounds treated with the larger dressing sizes of the PICO 7 system should generally be no more than 2cm (4/5 in.) in depth.

Exudate – PICO is intended for use on wounds where the level of exudate is low (up to 0.6g of liquid exudate/cm² of wound area/24 hours) to moderate (up to 1.1g of liquid exudate/cm² of wound area/24

Page 16 of 29

hours). 1g of exudate is approximately equal to 1ml of exudate. When used on a moderately exuding wound, the size of the wound should generally be no more than 25% of the dressing pad area.

8.2. Application

The dressing should only be applied by a healthcare professional.

- 1. Remove any excess hair to ensure close approximation of the dressing to the wound. If necessary, irrigate the wound with sterile saline and pat the wound dry.
- 2. Using a clean technique, peel off the first release handle and place the dressing centrally over the wound to reduce the chance of wound fluid coming into contact with the soft port. Ensure the dressing lies flat to the wound and the surrounding skin. The port should be uppermost from the wound (depending on the patient's primary position), placed on intact skin and not extending over the wound to prevent fluid pooling around

exudate, condition of the dressing, wound type/

size, orientation of the dressing, environmental

considerations or other patient considerations;

e.g. when PICO is used on infected wounds.

2. Inspect the PICO dressing regularly. If the dressing

press the orange button and disconnect the

appears ready for changing (see diagrams A-C),

dressing from the pump. The fixation strips should

be stretched away from the skin and the dressing

been fully removed. Apply another dressing as per

Section 8.2. Application, connect to the pump and

(A) Dressing properly positioned and

is acceptable to be left in place

Page 22 of 29

(B) Dressing requires change -

Port may block with fluid

(C) Dressing requires change -

Absorbent area is full

lifted at one corner and peeled back until it has

press the orange button to reinitiate the therapy.


Page 17 of 29

8.4. Use of PICO dressings

8.5. Use of PICO dressings non-adherent layers

Page 15 of 29

of the fixation strips (see Section 6. Precautions), wipe the area surrounding the dressing and allow skin to dry.

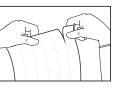

8. Apply the fixation strips all the way around the dressing border. Remove top carrier on the strip after each one has been applied. These strips maintain the seal over the wear time of the dressing. In awkward areas, it may be useful to apply the strips to help achieve a seal prior to switching on the pump. Place each strip so that it overlaps the dressing border by approximately 1cm (2/5 in.). Ensure tubing is not twisted or trapped between clothing.

Page 20 of 29

9. Showering and bathing

Light showering is permissible; however, the PICO pump should be disconnected (see Section 6. Precautions) and placed in a safe location where it will not get wet. The PICO dressing should not be exposed to a direct spray or submerged in water. Whilst disconnected, ensure the end of the tubing attached to the dressing is facing down so that water does not enter the tube.

10. General use

During use of your PICO dressing, for any information or troubleshooting on your pump refer to the respective PICO system user manual.



Please note that if at any time the fixation strips are removed, the dressing should also be replaced.

If desired, gel patches may be applied in addition to the fixation strips to help achieve or maintain a seal.

8.3. Dressing change Dressings should only be changed by a healthcare professional.

1. Dressings should only be changed in line with standard wound management guidelines, typically every 3-4 days. At the healthcare professional's discretion a PICO dressing may be left in place for up to 7 days. More frequent dressing changes may be required depending on the level of Page 21 of 29

clinical waste.

11. PICO dressing compatibility with other diagnostic procedures

PICO pumps and PICO dressings are compatible with defibrillation.

If in the event that defibrillation is required, remove the dressing and pump if it is positioned in a

location that will interfere with defibrillation. PICO dressings are MRI compatible, however, PICO

pump is not MRI compatible.

PICO dressings are not compatible with hyperbaric oxygen (HBO).

For compatibility information for the PICO pump, refer to the respective PICO system user manual.

12. Cautions

В

С

3. The PICO dressing should be disposed of as

This user manual is not intended as a guarantee or warranty. It is intended only as a guide. For medical questions please consult a physician.

The product must be used in accordance with this user manual and all applicable labeling.

PICO is packed in the UK with individual components being made in the following countries: Dressing - UK

Fixation strips - Origin as marked.





°Trade Marks of Smith & Nephew www.smith-nephew.com

[©]Smith & Nephew

This product may be covered by one or more US patents. See www.smith-nephew.com/patents Date of issue 03/2018

13. System variants

PICO Fluid Management Pack For use with PICO Single Use Ne Wound Therapy systems 5 x individually packaged dressin fixation strips 10cm x 20cm / 3.9in. x 7.9in. 10cm x 30cm / 3.9in. x 11.8in 10cm x 40cm / 3.9in. x 15.7in. 15cm x 15cm / 5.9in. x 5.9in. 15cm x 20cm / 5.9in. x 7.9in. 15cm x 30cm / 5.9in. x 11.8in. 20cm x 20cm / 7.9in. x 7.9in. 25cm x 25cm / 9.8in. x 9.8in. Multisite 15cm x 20cm / 5.9in. x 7.9in. Multisite 20cm x 25cm / 7.9in. x 9.8in.

66022020

66022021

Page 28 of 29

LOT Lot number.

Page 29 of 29

CE CE Mark.



Page 26 of 29

Page 27 of 29

	the soft port and blocking the the other remaining handle(s) and sn around the wound to prevent creat if required to ensure border is not	nooth the dressing asing. Reposition	 illuminate for 3 seconds. 5. Join the pump to the dressing tubes by twisting together the connectors. The soft tube can be directly connected to the pump if long tubing is not required.
	Once the dressing is in place, re and the batteries from the tray. Warning: The PICO 7 pu MAGNET. Keep the PIC least 4 inches (10 cm) other medical devices at all tim (See Section 5 Magnet Warning) The direction in which the batter be placed is indicated inside the	ump contains a CO 7 pump at away from nes. ries should	 6. Press the orange button to start the application of therapy. For any information or troubleshooting refer to the respective PICO system user manual.
	compartment. Insert the batterie cover. Following this all four indi	es. Replace the	7. If using NO-STING SKIN-PREP prior to application Page 19 of 29
Pl(an wc - 1 us ch clii Ga wc 8. Pl(nc sk inf	 4. Use of PICO dressings CO dressings are compatible with d foam fillers used in traditional in bound therapy where this is clinicated or example on a defect wound. We ded, the filler and the PICO dressing anged 2 to 3 times a week, acconnical protocol and manufacturer's auze should loosely fill to the surfound. Avoid over packing. 5. Use of PICO dressings of the protocol and manufacturer's auze should loosely fill to the surfound. Avoid over packing. CO dressings may be used over the n-adherent layer if required, for each or we for a dressings. 	n standard gauze negative pressure Nhen a filler is ng should be rding to local s instructions. face of the with the top of a example over a vounds at risk of	 8.6. Dressing removal Follow these instructions for dressing removal: 1. Stop the PICO pump by pressing the orange button - All indicators will turn off. 2. Remove the pump from the dressing by untwisting the connectors. 3. Remove the PICO dressing by stretching the fixation strips away from the skin. Lift the dressing at one corner and peel back until it has been fully removed. 4. Dispose of the PICO dressing and fixation strips as clinical waste.
		Page 23 of 29	Page 24 of 29
UNITED STATES Smith & Nephew, Inc., Smith & Nephew, Inc. 5600 Clearfork Main Street, Suite 600 Fort Worth, TX 76109 Customer Care Centre: 1-817-900-4000		k Main Street,	14. Glossary of symbols Warning: The PICO 7 pump contains a MAGNET that can cause other medical devices in close proximity to fail, leading to serious harm including death. The PICO 7 pump must be positioned at least 4 inches (10cm) away from other medical devices that could be affected by magnetic interference. See Section 5 Magnet Warning. R Caution: Federal (USA) law restricts this device to sale by or on order of a physician.
13. System variants PICO Fluid Management Pack For use with PICO Single Use Negative Pressure			Single Use. Do not reuse. Image: EU: Not for general waste.
W	ound Therapy systems		STERILE [E0] Product is sterilised by Ethylene Oxide.
	k individually packaged dressing ation strips	s, secondary	Caution.
-	cm x 20cm / 3.9in. x 7.9in.	66022022	Do not use if the package is damaged.
	cm x 30cm / 3.9in. x 11.8in.	66022023	Storage temperature.
	cm x 40cm / 3.9in. x 15.7in.	66022024	Manufacturer.
	cm x 15cm / 5.9in. x 5.9in.	66022025	Date of manufacture.
	cm x 20cm / 5.9in. x 7.9in. cm x 30cm / 5.9in. x 11.8in.	66022026 66022027	Consult instructions for use.
	11 x 30cm / 3.9m. x 1.0m.	66022027	Keep product out of sunlight.
	icm x 25cm / 9.8in. x 9.8in.	66022029	Keep dry.